

Magnide and Easy-slide application aids

for medical compression garments

Compression therapy

made easier



1 Fix the top of the loop to the top of the product by using the magnet.



3 Turn over the product and place it on a flat surface.



2 Pick up the side.



4 Fold in one side. Fold both sides with magnets to the center, so that the magnets make contact. The orange stitching is now visible on the outside.



5 Close the orange flap over the product so that the magnets make contact.



6 Open Toe



6 Closed Toe

Close the orange flap to the other side so that the magnets close.

Exclusively available from

Daylong
DIRECT.

Please use your phone to scan the QR codes for instructions on how to use the application aids



Magnide®



Easy-slide®

To find out more visit www.daylongdirect.com

For further support please contact us on:

Tel 0115 932 0144 • Email sales@daylongdirect.co.uk

ARION
SLIDE SOLUTIONS